

Ankle CCD

(Contracture Correction Device)

Patient's instructions for use

This is a custom-made medical device for the patient specified on page 1.

If you experience any problems with this device, please stop using it immediately and contact your Clinician using the contact details on page 1.



This device has been developed for the sole use of:

Patient Name

Hospital No.

NHS No.

As prescribed by:

Clinician Name

Company Name

Address

.....

Telephone No.

Safety

Inherent Risks:

This device has been developed in accordance to the UK Medical Device Regulations and is considered a low-risk medical device.

As a result, the risks to the user are very low. However, we would advise the following:

- Use the lever as shown by your Clinician.
- Do not force the lever beyond its travel limit.
- Be aware of the potential Finger Trap Point when using the lever. This is indicated on the device with a Yellow Trap Point Sticker...



- Do not wear or store the device next to a heat source (for example, radiator, gas fire, electric heater).

 Warning: Do not walk in this device.

Intended Purpose

This device has been developed to enable the following:

- Putting your CCD into a position that will help you apply it to your foot.
- Using the lever to help slowly apply the stretch.
- Allowing you to momentarily remove the stretch without needing to remove the device from your ankle. This might be particularly useful as you get used to it.

Note, this device is not intended to be worn when walking.

Cleaning & Maintenance

- This device has been predominantly manufactured using nylon. Some of the fasteners are made from coated mild steel and aluminium and may discolour over time if they come into contact with water for prolonged periods.
- You can clean the device down with mild detergent and warm water should it become necessary. Please allow it to fully dry before re-using.
- There is no requirement for ongoing maintenance, but should any aspects fail to work properly, please contact us to arrange a service/ check of the device.
- Your Clinician will check the mechanical workings at a routine review. Please let your Clinician know if you see or hear any abnormalities with your device in the meantime.